3Shape TRIOS®

Doctors with a TRIOS® scanner can send digital impressions directly to NDX D.H. Baker through 3Shape Communicate.

To send a file:

Dentists should be registered with a 3Shape Communicate account when the scanner is installed. If NDX D.H. Baker is not already pre-installed on the list of available labs, send a Connection Request from the Communicate account to NDX D.H. Baker.

Note: Dentists can register themselves at http://portal.3ShapeCommunicate.com. NDX D.H. Baker will respond that the lab is TRIOS ready to receive your scans.

- 1) After selecting the patient name, press New Session.
- 2) The lab list will appear, select NDX D.H. Baker.
- 3) Complete the order form information. In the Comments, specify specific materials, and if the case is model-less.
- 4) Complete the scanning process and press the Send Order envelope icon.
- 5) On the Send Order screen, verify the case information and scans, and then press Send Order.







3Shape TRIOS®

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX D.H. Baker, please contact us directly at **800.946.8880** or **di-DHBaker@NationalDentex.com.**