



Doctors with an iTero® scanner can send digital impressions directly to NDX Pacific through iTero online.

To send a file:

iTero Element:

- 1) Add **NDX Pacific** (ID# **18087**) by calling iTero Support at 800-577-8767.
- 2) Start a new Rx, and after entering patient name and case type, select *NDX Pacific* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the *Envelope icon*, add signature, and press *Confirm & Send*.

iTero HD2.9 and Previous Models

- 1) Add **NDX Pacific** (ID# **18087**) by calling iTero Support at 800-577-8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *NDX Pacific* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the *Send* button.



1601 N. Sepulveda Blvd #628 • Manhattan Beach, CA 90266 • 866.469.0305 • ndxpacific.com



NDX[®]  Pacific

iTero[®]

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Pacific, please contact us directly at **866.469.0305** or **sales@ndxpacific.com**.