3Shape TRIOS®

Doctors with a TRIOS® scanner can send digital impressions directly to NDX Green through 3Shape Communicate.

To send a file:

Dentists should be registered with a 3Shape Communicate account when the scanner is installed. If NDX Green is not already pre-installed on the list of available labs, send a Connection Request from the Communicate account to NDX Green.

Note: Dentists can register themselves at http://portal.3ShapeCommunicate.com. NDX Green will respond that the lab is TRIOS ready to receive your scans.

- 1) After selecting the patient name, press New Session.
- 2) The lab list will appear, select NDX Green.
- 3) Complete the order form information. In the Comments, specify specific materials, and if the case is model-less.
- 4) Complete the scanning process and press the Send Order envelope icon.
- 5) On the Send Order screen, verify the case information and scans, and then press Send Order.







3Shape TRIOS®

Should you have questions regarding the products, services and/or processes associated with submitting digital files to NDX Green, please contact us directly at **800.247.1365** or **digitalscans@greendentallab.com.**