

Full Arch Photogrammetry or Simpli-IOS[™] Case Records Checklist

Pre-Surgical Records

Must be sent to the lab at least two weeks before surgery

IOS Scans:

DENTATE PATIENTS

- ☐ Upper
- ☐ Lower
- ☐ Bite

EDENTULOUS PATIENTS

- ☐ Upper
- ☐ Lower
- ☐ Bite
- ☐ Soft tissue ridge scan
- ☐ 360° scans of the relined denture or relined bite block sent to the lab

Tip: If you are requesting a provisional denture, make sure the impressions or scans capture enough border detail for a denture to be manufactured.

Photos:

- ☐ Front Smiling
- ☐ Retracted

Tip: The retracted view shows enough teeth to have pre-op models aligned.

Day of Surgery Records

- ☐ Scan showing teeth before extractions with reference screws
- ☐ Scan of scan flags with the same reference screws
- ☐ Scan of putty tissue impression (if necessary)



SEND YOUR CASE

<https://nationaldentex.com/ndxmallowtru/send-a-case/full-arch-surgical-rx>